Budget Line

CLAMP RING

- en **Clamp Ring for sectional matrix system**
- nl **Klemring voor sectioneel matrixsysteem**
- fr **Bague de serrage pour système de matrice sectionnelle**
- de **Klemmring für Teilmatrizen**
- pl **Pierścień zaciskowy do systemów matryc sekcyjnych**
- da **Klemmering til sektionelle matrice-systemer**
- sv **Bygelklämma för sektionmatrissystem**
- no **Klemmering for seksjonsmatrisesystemer**
- fi **Kiinnitysrengas osista koottuja matriisijärjestelmiä varten**
- es **Anillo de sujeción para sistemas de matrices seccionales**
- pt **Anel de aperto para sistemas de matriz seccionada**
- it **Anello di ritenzione per sistemi a matrice sezionale**
- hr **Stezni prsten za sekcijske matrične sustave**
- sl **Vpenjalni obroč za sekcijske matrične sisteme**
- sk **Kofferdamový krúžok pre systémy sekčných matríc**
- cs **Kroužek pro sekční matrice**

Product # Description

63 09 12 Budget Line Clamp Ring, small

63 09 14 Budget Line Clamp Ring, medium

Polo MB Oisterwijk BV
Laarakkerweg 5
5061 JR Oisterwijk, The Netherlands

TANGSHAN ZHENG TONG EXHIBITION CO., LTD.
NO.24 Chaoyang Road, Lubei District,
Tangshan City, Hebei Province, China

EU REP SUNGO Cert GmbH.
Harffstraße 47,
40591 Düsseldorf, Germany

en

Indications:

With the function of fixing matrix bands, and separating tooth and good retention.

Composition:

Ring tips, Niti rings

Precautions:

1. It is used by dentists or health care professionals for oral treatment.
2. The force to open the rings shall not exceed the elastic limit of the Nitinol. The expansion shall not exceed 10 mm to avoid breakage.
3. Before use, the treatment area shall be covered with rubber dam to ensure safe use.
4. The use of rings requires placement through the forceps. The head of the forceps shall be placed at the special position of the rings and opened to prevent loosening.
5. Sterilization method: all models can be sterilized lasts for 5 minutes under high temperature (134°C) and high pressure.
6. Changing Ring tips:
Please replace the ring tips when it is damaged or cannot clamp tightly. First, take off the damaged tips by pliers, then put on the new tips in the right orientation. Finally, secure the ring tips in its final position with pliers.

Instructions for use:

1. Place the ring forceps tips into the special clamping part inside the ring, open the ring to the required gap width.
2. Put it into the tooth gap in the mouth, loosen the ring for compression and fix the matrix bands near to the tooth.
3. After the treatment, remove it with a forceps by opening the ring.

Storage:

Do not expose to the sun and avoid mixing with corrosive articles.

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