
Matrix Bands

STAINLESS TEEL

- en Matrix bands, Stainless steel
- nl Matrixbandjes, RVS
- fr Matrices, acier inoxydable
- de Matrizenbänder, Rostfreier Stahl
- pl Taśmy matrycowe, stal nierdzewna
- da Matricebånd, Rustfrit stål
- sv Matrisband, rostfritt stål
- no Matrisebånd, rustfritt stål
- fi Matrisinauhaut, ruostumaton teräs
- es Bandas matrices, acero inoxidable
- pt Matriz dentária, aço inoxidável
- it Bande matrice, acciaio inox
- hr Trake kalupa, nehrđajući čelik
- sl Tračne matrice, nerjaveče jeklo
- sk Matricové pásiky, chirurgická oceľ
- cs Páskové matrice, nerezová ocel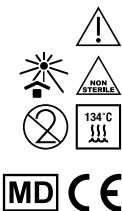

en Matrix bands, Stainless steel

Indications:

It plays the role of isolation or auxiliary restoration for oral class II cavity treatment.

Composition:

This product is made of thin stainless-steel sheet.

Instructions for use:

1. First select the appropriate model of band according to the cavity shape and insert the bands by placement forceps in the interdental gap at the edge of the tooth cavity.
2. Put in a suitable wedge, and clamp, make matrix band fit the tooth and avoid both leak and resin filled overhang.

Precautions:

1. This product is used for oral treatment by dentists or health care professionals.
2. If the surface of the matrix bands is found unclean during use, please stop using it.
3. Please pay attention to the matrix bands edge during use to prevent damage to mucosa and gums.
4. Disinfect or sterilize with high temperature and high pressure before use and note that the product packaging box cannot be sterilized with high temperature and high pressure. Dispose as medical waste after use.
5. This is a single-use device.

Storage:

Do not expose to the sun and avoid mixing with corrosive articles.

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