Thermafil®

EN	Instructions For Use	2
FR	Instructions d'utilisation	8
IT	Istruzioni per l'uso	15
DE	Gebrauchsanweisung	22
ES	Instrucciones de uso	29
PT	Instruções de Utilização	36
NL	Gebruiksaanwijzing	43
SV	Användarinstruktioner	50
DA	Brugsanvisning	56
NO	Brukerinstruksjoner	62
LT	Naudojimo instrukcija	68
LV	Lietošanas norādījumi	74
ET	Kasutusjuhised	80
CS	Návod k použití	87
SK	Návod na použitie	94
HU	Használati utasítás	101
RO	Instrucțiuni de utilizare	108
SR	Uputstva za upotrebu	115
BS	Uputstva za upotrebu	121
CNR	Uputstva za upotrebu	128
SQ	Udhëzimet për përdorim	135
EL	Οδηγίες χρήσης	142
TR	Kullanım Kılavuzu	149
KO	사용 지침	156

Thermafil®

PLASTIC CORE OBTURATORS

FOR DENTAL USE ONLY
FOR PROFESSIONAL USE ONLY
SINGLE-USE
NON-STERILE – TO BE DISINFECTED BEFORE USE

INSTRUCTIONS FOR USE

REF. A0166, A0167, A0168, A0348, A14110

1 INDICATIONS FOR USE

- Application field:
Endodontic obturation.
- Intended Purpose:
Thermafil® are intended to fill the shaped, cleaned and irrigated root canal space.
- Indication for use:
Thermafil® are indicated for treatment of endodontic disease.
- Patient Population:
Patients who need root canal treatment.
- Intended User:
Endodontic devices are to be used only in a clinical or hospital environment, following good dental practice, by qualified dental professionals such as general practitioners as well as Endo specialists (Endodontist) and Dental Assistants.

2 CONTRAINDICATIONS

This product may contain traces of dry natural rubber. Thermafil® shall not be used on patients with known allergies to Latex.

3 DELIVERY FORMS

Code	Product	Packaging and Content
A0166	Thermafil ® Obturators	Plastic box of 6
A0167	Thermafil ® Obturators Assorted Posterior Kit	Plastic box of 20
A0168	Thermafil ® Obturators Assorted Anterior Kit	Plastic box of 20
A0348	Thermafil ® Obturators	Plastic box of 30
A14110	Thermafil® for ProTaper Universal	Plastic box of 6 Plastic box of 20

4 PHYSICAL AND CHEMICAL PROPERTIES

- Radiopacity: Equivalent to 5 mm of aluminum or more in thickness
- Gutta-Percha Melting Temperature: <60°C

5 COMPOSITION

The following components are applicable to all obturators

Gutta-Percha surrounding layer components:

- Zinc Oxide (<60%)
- Synthetic gutta-percha (<25%)
- Barium sulfate (<20%)
- Titanium dioxide (<5%)
- Orange colorant (<2%)
- Red Colorant (<2%)
- Antioxidant (<1%)
- Inert fillers (<10%)

Thermafil® inner carrier components:

- Liquid Crystal Polymer or Polysulfone (35 - 65%)
- Tungsten (35 - 65%)

Silicone stopper

- Silicone rubber

6 COMPATIBLE DEVICES

These devices are used in combination with:

- ThermaPrep® which is a heating equipment used to heat the obturators prior to obturation.
- Size verifiers REF. A0175 are used to verify the size of a root canal before obturation.
- Post-Space Burs REF. A0051 are used to eliminate the coronal part of the filling.
- Therma-Cut® burs REF. A0050 are used to cut the head of obturators once placed at the working length.
- Endodontic sealers, such as AH Plus® and AH Plus® Bioceramic Sealer, are used to coat root canal walls and remains at the interface between the root canal and the gutta-percha material.

7 WARNINGS

- Thermafil® obturators are intended for single use only. Single-use products shall not be processed and reused. Reusing these products increases the risk of cross-contamination and/or breakage.
- Thermafil® obturators may contain traces of dry natural rubber. It is not recommended for use in patients that are allergic to natural latex rubber.
- Use of the devices might lead to the following residual risks:

- Dissatisfaction of the user and delay in treatment / Patient discomfort.
- Retreatment (initial infection never healed).
- Serious infection / Serious health consequences for the patient (e.g. biological hazards such as infection, toxic or allergic reaction for the patient) have never been reported and are very unlikely but cannot be totally excluded.

8 PRECAUTIONS

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Inspect the packaging and product before use: if the packaging or the product is damaged, do not use the product and discard it.
- Use a rubber dam system during the endodontic procedure.
- For your own safety, wear personal protective equipment (gloves, glasses, mask).
- Products shall be disposed of according to local regulations for the safe disposal of sharp and contaminated devices.
- Thermafil® obturators shall not to be sterilized.
- Do not use disinfecting solutions containing Phenol or any products which are not compatible with the treated filling material.
- Select the Thermafil® obturators according to the appropriate size verifier that fits the shaped canal passively. Thermafil® obturators and/or hand files should not be used as size verifiers.
- Do not remove gutta-percha from the apical section of the obturator prior to filling the root canal space. Removing the gutta-percha may damage the obturator.

9 ADVERSE REACTIONS / UNDESIRABLE SIDE-EFFECTS

Following undesirable side effects may occur:

- Patients with latex sensitivity may experience allergic reactions to Thermafil® obturators; due to potential presence of dry natural rubber.
- Temporary post-operative discomfort could arise generally as a result of receiving a root canal treatment.

10 STORAGE CONDITIONS

- Keep away from sunlight and heat.
- Maintain under room temperature ($\leq 30^{\circ}\text{C}$ (86°F)).
- Store in a dry area.

11 STEP-BY-STEP INSTRUCTIONS

The obturators shall only be used in a clinical or hospital environment, by qualified dental professionals.

- 1) After shaping and cleaning the root canal, obturation technique must be set by the professional.

- 2) Flare the canal coronally and ensure the Thermafil® obturator has sufficient space to enter the root canal along the straight-line access by using an orifice shaping instrument as necessary.
- 3) In order to determine the size of the Thermafil® which will be used for the filling, a size verifier corresponding in size to the last instrument used up to the working length is employed to verify the size of the root canal.
This verifier must:
 - Slip in easily up to the working length without force (if the verifier does not reach the working length, increase the taper of the preparation or use a smaller size verifier).
 - Present a slight sensation of resistance on withdrawal (if there is no sensation of resistance on withdrawal, try a larger size verifier).
- 4) Select the Thermafil® obturator corresponding to the size verifier which adapts itself passively to the working length.
 - The depth references and/or silicon stop must be used to determine the distance of the working length on the Thermafil® central plastic carrier.
- 5) Disinfect the obturator as per Hygiene, disinfection, cleaning, and sterilization [► 6] section.
- 6) Drying:
The shaped and cleaned root canal shall be dried using sterile absorbent paper points of matching size.
- 7) Sealer:
Endodontic sealer shall be applied following its Instructions for Use (<https://www.dentsplysirona.com/ifu>). Any excess sealer that may have accumulated on the canal walls or pooled in the apex shall be removed.
- 8) The obturators are heated in the ThermaPrep® obturator oven.
 - For details on the obturator oven please refer to the obturator oven instruction manual. (<https://www.dentsplysirona.com/ifu>)
 - Put both obturator holders in the upper position by pressing the back of the holder (marked area).
 - After disinfecting the obturator and setting the rubber stop, place the obturator in one of the obturator holders, lightly press on top to have it firmly seating.
 - Regardless of what working length you have determined; the rubber stop must be under the holder.
 - Make sure the obturator is hanging straight, not at an angle. The obturator must clear all sides of the heating chamber.
 - Push the button which corresponds to the selected obturator holder and select the size of the obturator you wish to heat.
 - Repeatedly push this button to select the heat setting. Refer to the obturator oven instruction manual for heat settings.
 - To start the heating process, push the obturator holder down until you hear it click. It will raise a little but keep in a steady down position.
 - After the first signal “beep” of the ThermaPrep® oven, the obturator is ready for use.
 - Push the back of the obturator holder (marked area) to let it raise slowly.

- The indicator light around the button will go off.
 - Take the obturator carefully out of the holder by pulling it towards you, making sure not to scrape the obturator on any part of the holder.
 - The oven will “beep” every 15 seconds to remind you that the obturator is still in the oven. After 90 seconds, the oven will switch off automatically.
- 9) Carefully extract the obturator from the oven and insert it immediately into the root canal with a slow, firm, and continuous movement in the apical direction.
- 10) Ensure that the movement is always in the direction of the carrier to prevent it from bending.
- On the insertion, the gutta-percha apically precedes the carrier, thus blocking the root canal towards the apex whilst filling all the lateral root canals in a single insertion.
- 11) The pressure should be maintained a few seconds after the apex has been reached.
- If several root canals have to be filled, use the left and right supports alternately in order to save time.
- 12) Take a radiograph to confirm that the root canal is filled up to working length.
- While stabilizing the Obturator with your index finger, remove the shaft and handle at the orifice with a Thermo-Cut® bur with a turbine without spray, or use a sharp spoon excavator.
- 13) Using an endodontic excavator, remove the surplus gutta-percha which could block access to the other root canals.
- It is very important at this stage to use a plugger to condense the softened gutta in the coronal part around the carrier.
- 14) Repeat all the steps detailed above for all the root canals of a pluri-radicular tooth.
- When filling several root canals of a molar, always start with the root canal having the most difficult access.
- 15) Dispose of obturators in an appropriate biohazard container.

12 HYGIENE, DISINFECTION, CLEANING, AND STERILIZATION

- Obturators are intended for Single use only.
- Obturators are non sterilizable device, please follow the below instructions for Disinfection:
 - Immerse the obturators in 5.25% NaOCl for 1 to 5 minutes at ambient temperature.
 - Remove all the bubbles from the surface of the obturators.
 - Gently wipe the Thermoafil® with sterile gauze moistened with 70% isopropyl alcohol.
 - Let the obturators air dry.
 - Do not use disinfecting solutions containing Phenol or any products which are not compatible with gutta-percha.
- For other products listed in this IFU, please follow processing instructions from the manufacturer.
 - For Post-Space Burs REF. A0051, Thermo-Cut® burs REF. A0050 and Size verifiers REF. A0175, please follow the Dentsply Sirona General Processing Instructions (<https://www.dentsplysirona.com/ifu>) for the disinfection, cleaning and sterilization steps.

13 SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Information is available at both the European database on medical devices (EUDAMED) and the IFU website, where it is linked to the Basic UDI-DI.

EUDAMED

<https://ec.europa.eu/tools/eudamed>

IFU

www.dentsplysirona.com/ifu

The Basic UDI-DI number is: ++J00310044DZ

14 ADDITIONAL INFORMATION

- To get a free printed copy of IFU please see section “Get a print copy of IFU” on website <https://www.dentsplysirona.com/en/service-contact/ifu.html>
- Explanation of non-harmonized symbols for IFUs and labels, see IFU Symbols (<https://www.dentsplysirona.com/en/service-contact/ifu/ifu-symbols.html>).

15 LOT NUMBER AND EXPIRATION DATE

Do not use after expiration date.

16 SERIOUS INCIDENT REPORTING

- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

DISCLAIMER

Users are solely responsible for any deviation from these instructions, and/or the use of alternative methods for processing. Dentsply Sirona accepts no liability for damage, injury, or any legal responsibility incurred directly or indirectly by the user due to a deviation from the instructions for use set forth below. The user shall observe safe and lawful practices including, but not limited to, those set forth in this document.

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DENTSPLY DeTrey GmbH