ProRoot® Bio Sealer

Bioceramic Endo Sealer

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Bioceramic Endo Sealer

USA: Rx only.

Content	Page
1 Composition & description	2
2 Material characteristics	2
3 Indication	2
4 Contraindications	2
5 Adverse reactions	3
6 Warnings	3
7 Interactions with other dental materials	3
8 Storage	3
9 Precautions	3
10 Clinical directions for use	4
A Addendum ProRoot[®] Bio Sealer	83

1 Composition & description

Bioactive paste consisting of an extremely fine, inorganic powder of tricalcium/dicalcium silicate in an organic medium. The product is packaged ready-to-use. No mixing is required. ProRoot[®] Bio Sealer is designed to set in vivo in the presence of moisture provided by the surrounding tissues.

2 Material characteristics

- Bioactive bioceramic
- Does not discolor teeth
- Radiopaque
- Resin-free

3 Indication

Obturation and sealing of root canals.

4 Contraindications

- Hypersensitivity against caustic (high pH) solutions.
- Do not use for primary tooth pulpectomy (root canal filling) unless the permanent successor tooth is absent.

5 Adverse reactions

Reversible acute inflammation of the oral mucosa if contacted with the unset paste.

6 Warnings

ProRoot® Bio Sealer is caustic, as are all tricalcium silicates.

7 Interactions with other dental materials

None known.

8 Storage

Store at room temperature (DO NOT REFRIGERATE). To prevent hardening of the ProRoot® Bio Sealer, immediately recap after each use. Store the syringe in the protective aluminum packaging provided.

9 Precautions

- Do not overfill the root canals. When a large amount of material is overfilled in the mandibular canal (inferior alveolar canal), immediate surgical removal of the material should be considered, as with all root canal materials, according to state-of-the-art policy.
- Minimize overextension of the material beyond the apex.
- Avoid contact of unset sealer with skin or oral mucosa. After incidental contact, wash and rinse with water.
- Wear suitable gloves and protective glasses during use.
- ProRoot® Bio Sealer MUST BE KEPT WELL SEALED. Immediately recap after each use.
- To protect against moisture intrusion, store ProRoot® Bio Sealer in the protective aluminum packaging provided.
- Always use a new tip when applying ProRoot® Bio Sealer.
- Avoid touching the syringe to a contaminated surface.
- Cover the syringe body with a disposable protective sleeve if used intraorally, to minimize contamination of the syringe.
- ProRoot® Bio Sealer and tips are provided in clean non-sterile packaging. These products cannot be sterilized. Clinicians should follow their established protocols for cleaning and disinfection of the ProRoot® Bio Sealer syringe between uses.

See: <http://www.cdc.gov/infection-control/media/pdfs/Guideline-Disinfection-H.pdf>

- Setting of tricalcium silicates is inhibited in acidic environments such as infected sites.

10 Clinical directions for use

- a) Excavate carious tooth structure using a round bur in a handpiece at low speed or use hand instruments.
- b) Debride, clean, shape and disinfect the root canal system using intra-canal instruments under rubber dam isolation.
- c) Irrigate the root canal system, including removal of smear layer. Perform final irrigation with distilled water and dry the root canal using paper points. Moisture is required for setting.
- d) Verify uniform flow prior to applying intraorally by extruding a small amount of material onto a dispensing pad.
- e) ProRoot® Bio Sealer may be used with or without the Tip, but the Tip is recommended.

USE WITHOUT the Tip

- For obturation techniques where most of the canal is obturated by gutta percha points, first apply a light coating of ProRoot® Bio Sealer material to the canal walls using a device of your choice.
- Second, coat the disinfected and dried gutta percha points with ProRoot® Bio Sealer material before insertion.

USING the Tip

- Attach the tip to the ProRoot® Bio Sealer syringe.
- Insert the tip into the canal to the desired depth, taking care to not insert the end past the middle third of the root canal.
- Do not force the tip into the canal such that it seals against the canal walls. Doing so may force the sealer to be expressed beyond the apex.
- Apply gentle, steady pressure to the syringe plunger with a constant outward movement from the canal as the material is injected.
- If using a higher gauge (smaller diameter) dispensing tip, pump the syringe until ProRoot® Bio Sealer is seen dispensing. Apply constant pressure to continue with the dispensing.

NOTE: ProRoot® Bio Sealer may be difficult to dispense using higher gauge (smaller diameter) tips.

- f) Slowly, insert a single or multiple cones into the canal.
- g) Melt and cut off excess gutta percha at the orifice following the manufacturer's directions.
- h) Remove excess sealer with a moist cotton pellet.
- i) Confirm placement of the material in the complete root canal system and check for any over-extrusion with a radiograph.

NOTE: For removal of root canal filling, use conventional instrumentation to remove set sealer, including hand/rotary files and ultrasonic instrument tips, as with any other sealer.

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A Addendum ProRoot® Bio Sealer

ADA 57, ISO 6876

Test	Acceptance Criteria	ProRoot® Bio Sealer Results
Working time at room temperature	None	~ 30 minutes
Setting time at 37 °C, in vivo (or moist environment)	Material must set	Meets ISO 6876 standard
Flow	> 17 mm	Meets ISO 6876 standard
Film Thickness	< 50 µm	< 50 µm
Solubility	< 3% by weight	< 3% by weight
Dimensional stability	< 1% shrinkage & < 0.1% expansion	~ 0.3% shrinkage
Radiopacity	> 3 mm equivalent of Al	6 mm of Al

Symbols used on labeling

	Manufacturer
	Authorized Representative in the European Community
	Prescription Only
	Consult Instructions for Use
	Caution
	Keep Dry
	Lot Number
	Catalog Number
	Expiration Date
	Irritant
	Medical Device

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MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Manufactured by
NuSmile, Ltd
3315 West 12th St
Houston, TX 77008
USA

Imported by
Dentsply Sirona Europe GmbH
Sirona Str. 1
5071 Wals bei Salzburg
AUSTRIA
www.dentsplysirona.com

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MDSS GmbH
MDSS-UK RP Ltd.