# TEETHMATE™ DESENSITIZER

<b>EN</b>	ENGLISH	English	TOOTH DESENSITIZING MEDICAL DEVICE
<b>FR</b>	FRANÇAIS	French	DISPOSITIF MÉDICAL DE DÉSENSIBILISATION DES DENTS
<b>ES</b>	ESPAÑOL	Spanish	DISPOSITIVO MÉDICO PARA LA DESENSIBILIZACIÓN DEL DIENTE
<b>IT</b>	ITALIANO	Italian	DISPOSITIVO MEDICO PER DESENSIBILIZZAZIONE DENTALE
<b>NL</b>	NEDERLANDS	Dutch	MEDISCH PRODUCT VOOR DE DESENSIBILISERING VAN TANDEN
<b>DE</b>	DEUTSCH	German	MEDIZINPRODUKT ZUR DESENSIBILISIERUNG VON ZÄHNEN
<b>SV</b>	SVENSKA	Swedish	MEDICINTEKNISK PRODUKT FÖR TANDDESENSIBILISERING
<b>NO</b>	NORSK	Norwegian	MEDISINPRODUKT FOR DESENSIBILISERING AV TENNER
<b>FI</b>	SUOMI	Finnish	HAMPAIDEN HERKKYYTTÄ VÄHENTÄVÄ LÄÄKINNÄLLINEN LAITE
<b>DA</b>	DANSK	Danish	MEDICINSK UDSTYR TIL DESENSIBILISERING AF TÆNDER
<b>PT</b>	PORTUGUÊS	Portuguese	DISPOSITIVO MÉDICO PARA DESSENSIBILIZAÇÃO DENTÁRIA
<b>EL</b>	ΕΛΛΗΝΙΚΑ	Greek	ΙΑΤΡΙΚΗ ΣΥΣΚΕΥΗ ΑΠΕΥΑΙΣΘΗΤΟΠΟΙΗΣΗΣ ΟΔΟΝΤΩΝ
<b>TR</b>	TÜRKÇE	Turkish	DİŞ HASSASİYET GİDERİCİ TIBBİ CİHAZ
<b>PL</b>	POLSKI	Polish	WYRÓB MEDYCZNY DO LECZENIA NADWRAŻLIWOŚCI ZĘBINY
<b>RO</b>	ROMÂNĂ	Romanian	DISPOZITIV MEDICAL PENTRU DESENSIBILIZAREA DINTELUI
<b>HR</b>	HRVATSKI	Croatian	MEDICINSKI UREĐAJ ZA DESENZIBILIZACIJU ZUBA
<b>HU</b>	MAGYAR	Hungarian	FOGÉRZÉKENYSÉG-CSÖKKENTŐ ORVOSTECHNIKAI ESZKÖZ
<b>SL</b>	SLOVENŠČINA	Slovenian	MEDICINSKI IZDELEK ZA ZMANJŠANJE OBČUTLJIVOSTI ZOB
<b>CS</b>	ČESKY	Czech	LÉKAŘSKÝ PRODUKT PRO DESENZIBILIZACI ZUBŮ
<b>SK</b>	SLOVENSKY	Slovak	LEKÁRSKE ZARIADENIE NA ZNECITLIVENIE ZUBOV
<b>BG</b>	БЪЛГАРСКИ	Bulgarian	МЕДИЦИНСКО ИЗДЕЛИЕ ЗА ДЕЗИНСИБИЛИЗАЦИЯ НА ЗЪБИ
<b>UK</b>	УКРАЇНСЬКА	Ukrainian	МЕДИЧНИЙ ПРИСТРІЙ ДЛЯ ДЕСЕНСІБІЛІЗАЦІЇ ЗУБІВ
<b>ET</b>	EESTI	Estonian	MEDITSIINITOODE HAMMASTE TUNDLIKKUSE VÄHENDAMISEKS
<b>LV</b>	LATVIEŠU	Latvian	ZOBĀRSTNIECĪBAS PRODUKTS ZOBU DESENSIBILIZĀCIJAI
<b>LT</b>	LIETUVIŠKAI	Lithuanian	DANTŲ NUJAUTRINIMO MEDICINOS PRIETAISAS

**Kuraray Noritake Dental Inc.**

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

**EMERGO EUROPE**

Westervoortsedijk 60,  
6827 AT Arnhem, The Netherlands

**Kuraray Europe GmbH (EU Importer)**

Philipp-Reis-Str. 4,  
65795 Hattersheim am Main, Germany

Phone:+49 (0)69 305 35835 Fax:+49 (0)69 305 98 35835

URL:<https://www.kuraraynoritake.eu>

# TEETHMATE™ DESENSITIZER

## I. INTRODUCTION

TEETHMATE DESENSITIZER is composed of POWDER which contains Calcium Phosphate as a major component and LIQUID which mainly contains water. POWDER and LIQUID are mixed to obtain slurry. The slurry applied to the affected area transforms to a hardened material of hydroxyapatite and it suppresses hypersensitivity by sealing the dentin tubules and microcracks in the enamel. The general clinical benefit of this product is to restore tooth function for the following INDICATIONS FOR USE.

## II. INDICATIONS FOR USE

TEETHMATE DESENSITIZER is indicated for reduction of tooth hypersensitivity by the following treatments:

- [1] Treatment of dentin exposed by toothbrush abrasion, gingival recession, periodontal disease and/or acid erosion
- [2] Treatment of dentin after mechanical tooth cleaning, scaling and/or root planing
- [3] Treatment of tooth surface before and/or after bleaching
- [4] Treatment of prepared dentin for fillings and/or prosthetic restorations

## III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to this product

## IV. PRECAUTIONS

### 1. Safety precautions

1. This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to any of the components.
2. If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching or numbness, discontinue use of the product and seek medical attention.
3. Avoid direct contact with the skin and/or soft tissue to prevent hypersensitivity. Wear gloves or take appropriate precautions when using the product.
4. Use caution to prevent the product from getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
5. If the product gets in the eye, immediately wash the eye with copious amounts of water and consult a physician.
6. Use caution to prevent the patient from accidentally swallowing the product.
7. Do not reuse the applicator brush and the interdental brush to avoid cross contamination. Discard them after use.
8. The patient should be instructed to avoid drinking or eating for 45 minutes after the completion of treatment with the product. Or the sealing material constructed by the product may drop out from the dentin tubules and the sealing effect may decrease.
9. This product does not permanently suppress hypersensitivity. If the hypersensitivity recurs, repeat treatment with this product.
10. Dispose of this product as a medical waste to prevent infection.

### 2. Handling and manipulation precautions

1. The product must not be used for any purposes other than specified in [II.INDICATIONS FOR USE].
2. The use of this product is restricted to dental professionals.
3. Use a pulp capping agent in a cavity close to the pulp.
4. Do not mix the product with any other dental materials.
5. If the instrument or the container of this product is damaged, take care of yourself from any damages and immediately discontinue use.
6. Shake the powder container gently before measuring. Measure a level spoonful of POWDER by using the inner cap edge.
7. After dispensing LIQUID or measuring POWDER, each container should be stoppered tightly as soon as possible. In particular, in the powder container, take care not to allow moisture infiltrating because the powder component reacts to water to harden.
8. If the hardened material is found in the powder container due to reacted to water, remove the hardened part before use.
9. When dispensing the liquid component into the mixing dish, hold the container completely vertically. Or the correct amount of LIQUID is not dispensed.
10. Redispense after thoroughly wiping away the formerly dispensed LIQUID from the mixing dish if air bubbles are found in it.
11. The mixing ratio of LIQUID/POWDER must be accordance with the table "Standard mixing ratio" in "A-2" of [VI.CLINICAL PROCEDURES]. Excessive LIQUID will impair the performance of sealing the dentin tubules.
12. The mixing of LIQUID/POWDER must be continued for more than 15 seconds. Insufficient mixing time causes unfavorable slurry to apply.

13. Use the slurry within 10 minutes after the completion of mixing. The viscosity of slurry will increase with time, making it difficult to apply.
14. After each use, any residues of POWDER on the measuring spoon must be thoroughly wiped away.
15. Dentin tubules will not be sealed sufficiently simply by placing the slurry on the affected area. Make sure to rub the slurry on the surface of the affected area.
16. Do not use other than applicator brush, cotton pellet or interdental brush when rubbing the slurry into the objective area.
17. In case of treatment of prepared dentin for fillings and/or prosthetic restorations, clean the prepared dentin and remove minute remaining particles of slurry to avoid loss of adhesive strength that is assumed by the relevant bonding material.

## 3. Storage precautions

1. The product must be used by the expiration date indicated on the package.
2. The product must be stored at 2 - 25°C / 36 -77°F when not in use. When the powder container is stored in the refrigerator, it should be left unopened at room temperature for more than 15 minutes after taking out from the refrigerator, before each use. Less exposure time at room temperature may cause condensation on the inside of the powder container.
3. The product must be kept away from extreme heat or direct sunlight.
4. The product must be stored in a proper place where only dental practitioners can access.

## V. COMPONENTS

Please see the outside of the package for contents and quantity.

### Principal ingredients

- 1) POWDER
  - Tetracalcium phosphate (55-75%)
  - Dicalcium phosphate anhydrous (10-35%)
  - Disodium hydrogen phosphate anhydrous (< 15%)
  - Colloidal silica (< 5%)
- 2) LIQUID
  - Water (99.9%)
  - Preservative (< 0.1%)

Units in parentheses are mass %.

- 3) Accessories
    - Mixing dish\*
    - Measuring spoon
    - Applicator brush (superfine <white>)
- \*Consumables

## VI. CLINICAL PROCEDURES

### A. Standard procedure I (INDICATIONS FOR USE [1] to [3])

- [1] Treatment of dentin exposed by toothbrush abrasion, gingival recession, periodontal disease and/or acid erosion
- [2] Treatment of dentin after mechanical tooth cleaning, scaling and/or root planing
- [3] Treatment of tooth surface before and/or after bleaching

#### A-1. Tooth surface cleaning

Clean the tooth surface in the usual manner. Wipe away saliva and plaque from the affected area using cotton pellets etc. If the affected area is severely contaminated with plaque or stain, use a suitable tooth surface abrasive. After tooth surface cleaning, the drying of the affected area by air blowing is not necessary.

#### A-2. Mixing procedure

Standard mixing ratio

Objective area	LIQUID	POWDER
1 tooth	1 drop	1 spoonful (small measuring spoon)
2 or 3 teeth	2 drops	1 spoonful (large measuring spoon)

- (1) Hold the liquid container vertically over the mixing dish and dispense the intended drops of LIQUID into the dish.
- (2) Shake the powder container gently before measuring. Using the measuring spoon, measure a level spoonful of POWDER by using the inner cap edge. And put it on the formerly dispensed LIQUID in the mixing dish.
- (3) Mix LIQUID and POWDER for more than 15 seconds to make the slurry, using the applicator brush.

#### A-3. Application of the slurry

Apply the slurry to the affected area using the applicator brush and then let it penetrate sufficiently by rubbing the area for more than 30 seconds. A cotton pellet can be used instead of applicator brush for large affected areas. If the applicator brush does not work properly in interdental spaces, an interdental brush can be used for this purpose.

**A-4. Rinsing**

Rinse the excess slurry with water spray or by having the patient rinse.

**A-5. Checking of the desensitizing effect**

Check the effect of treatment by blowing air onto the affected area.

\* If dentin hypersensitivity persists, repeat steps A-3, A-4 and A-5.

**B. Standard procedure II (INDICATIONS FOR USE [4])**

[4] Treatment of prepared dentin for fillings and/or prosthetic restorations

**B-1. Preparing cavity or abutment tooth**

Using the usual manner, prepare, clean and disinfect the cavity or abutment tooth.

**B-2. Mixing procedure**

Use the same procedure described in "A-2".

**B-3. Application of the slurry**

Use the same procedure described in "A-3".

**B-4. Rinsing**

Use the same procedure described in "A-4".

**B-5. Checking of the desensitizing effect**

Use the same procedure described in "A-5".

**B-6. Cleaning prepared dentin**

Clean the prepared dentin for more than 10 seconds using cotton pellets moistened with water to remove the minute remaining particles of the slurry from the dentin surface.

**B-7. Application of dental adhesive for filling material or cementation material for prosthetic restoration**

Follow the manufacturer's recommended procedure for each dental adhesive or cementation material.

Note: When using dental adhesives or cementation materials which require prior phosphoric acid etching (ex: CLEARFIL PHOTO BOND), the etching step should be performed in the usual manner before applying this product.

**[WARRANTY]**

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

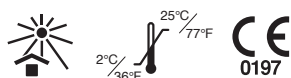
**[NOTE]**

If a serious incident attributable to this product occurs, report it to the manufacturer's authorized representative shown below and the regulatory authorities of the country in which the user/patient resides.

**[NOTE]**

"CLEARFIL" and "TEETHMATE" are registered trademarks or trademarks of KURARAY CO., LTD.

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„CLEARFIL“ ir „TEETHMATE“ yra „KURARAY CO., LTD“ registruotieji prekės ženklai arba prekės ženklai.