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Daily Oil Plus

EN

UNIVERSAL LUBRICATING SPRAY FOR THE MAINTENANCE OF ROTATING INSTRUMENTS, TURBINES, HANDPIECES, CONTRAANGLES AND MOTORS.

MEDICAL DEVICE FOR PROFESSIONAL USE ONLY.

Class I medical device according to Regulation (EU) 2017/745 on Medical Devices.

Apply daily and before autoclave sterilising, lubricate with two short sprays according to the Instructions for Use of the instrument.

Run the equipment at idle and remove the product in excess from the external surfaces.

Do not use after the expiry date stated on the package.

Store in a cool and dry place away from heat, light and damp.

Do not use if the packaging is not intact.

For operators in Europe: report any serious incident occurring in relation to the device to CEFLA s.c. and to the competent authority of the Member State in which the user and/or patient is located.

CONTAINS: liquefied propane/butane gas, synthetic lubricant.

DANGER:

H222 Extremely flammable aerosol.

H229 Pressurized container: May burst if heated.

P210 Keep away from heat/sparks/open flames/hot surfaces. No smoking.

P211 Do not spray on an open flame or other ignition source.

EUH066 Repeated exposure may cause skin dryness or cracking.

P251 Do not pierce or burn, even after use.

P403+P235 Store in a well ventilated place. Keep cool.

P410+P412 Protect from direct sunlight. Do not expose to temperatures exceeding 50°C/122°F.

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