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Notes on application:
Phosphoric acid etching gel for the conditioning of dental enamel and dentin in dentistry.

Safety instructions:
H 314 Causes severe skin burns and eye damage. Wear protective gloves and eye protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do so. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. IF SWALLOWED: Rinse mouth. DO NOT induce vomiting IF ON SKIN: Wash with plenty of soap and water. Contains ortho-phosphoric acid

Product description:

The PolyDent etching gel based on 35/38 % phosphoric acid is characterised by optimal thixotropic flow behaviour. It is used for etching tooth substance within the framework of the enamel etching technique and total-etch technique. According to current knowledge, the product has no side effects if the safety and application instructions are observed. Underfilling materials can be dissolved by the etching gel during longer exposure times.

- For dental use only!
- Do not use beyond the shelf life date
- Close refill syringe immediately after use!

Applicator syringe and applicator needles can only be used once.

Instructions for use:
Filling the empty applicator syringe:
 An empty applicator syringe is attached to the supply syringe using the adapter (turn 180° clockwise). The small syringe is filled by pressing the large syringe plunger down carefully and slowly. If necessary, simultaneously pull the small syringe plunger a little and make sure that the small syringe is not overfilled or the plunger is pressed out of the syringe!
 Then, the small applicator syringe is twisted off the adapter and the 50 ml-syringe is closed tightly. A cannula is screwed onto the applicator syringe in a clockwise motion and tightened. The etching gel can now be applied by carefully pressing the syringe plunger. The filled applicator syringe and the application cannula are intended for single use on a patient; the syringe must not be refilled.
 When using accessory applicator syringes, the shelf life date and batch number of the refill syringe must be transferred manually to the label of the filled applicator syringe.

Preparation:

The areas of the tooth substance to be etched must be clearly delimited.

- 1) The use of a rubber dam is recommended! Any contact with soft tissue (skin, oral mucosa, eyes etc.) is to be avoided. If etching gets onto the soft tissue, immediately rinse the area with plenty of water and, if necessary, contact a doctor.
- 2) Before preparation, clean the tooth using a rubber cup and fluoride- and fat-free polishing paste.
- 3) Use water to properly remove cleaning residues, clean, and dry the surface of the tooth with oil-free compressed air.
- 4) Prepare the cavity accordingly; bevel the melted edges to achieve permanent enamel junctions. Rinse the cavity thoroughly with water and with oil-free compressed air. Avoid overdrilling!
- 5) Protect dental pulp in deep cavities (e.g. cover with a thin layer of suitable underfilling material). The cavity should be dry and free of contamination, whereby the overdrilling of the dentine is to be prevented.

Etching process

- 6) Apply etching process to the enamel margins of the cavity and, if necessary, to the dentin.

Recommended application times:

- 15 - 30 sec. on tooth enamel
- 10 - 15 sec. on dentin
- 50 - 80 sec. on milk teeth and teeth with a high content of fluoride

- 7) Rinse with water for at least 20 seconds, no etching gel should be left.
- 8) Dry with oil- and water-free compressed air. The etched area must not be touched or contaminated by saliva until the treatment is completed. However, should contamination occur after the etching and drying, the area must be etched again briefly (approx. 10 sec.). The etched area should look chalky white! After etching, you can continue with the application of adhesives etc. in accordance with the manufacturer's instructions for use.

Smaller quantities of etching gel diluted with water can be disposed of via the sewage system. Dispose of uncleaned packaging according to local regulations.

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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