Budget Line

SECTIONAL MATRIX BANDS

- en **Pre-contoured matrix**
 nl **Voorgevormde matrixbandjes**
 fr **Matrices préformées**
 de **Vorgeformte Matrizenbänder**
 pl **Wstępnie uformowane taśmy matrycowe**
 da **Præformede Matricebånd**
 sv **Förformade Matrisband**
 no **Forformede Matrisebånd**
 fi **Valmiiksi muotoillut Matriisinauhat**
 es **Bandas matrices preformadas**
 pt **Matriz dentária pré-moldadas**
 it **Bande matrice preformate**
 hr **Trake kalupa preformirano**
 sl **Predhodno oblikovani trakovi matrice**
 sk **Matricové pásiky predtvarovaný**
 cs **Predtvarované páskové matrice**

Product #	Description
63 09 02	Sectional Matrix Bands, 3.5 mm
63 09 04	Sectional Matrix Bands, 4.5 mm
63 09 06	Sectional Matrix Bands, 5.5 mm
63 09 08	Sectional Matrix Bands, 6.5 mm

Polo MB Oisterwijk BV
 Laarakkerweg 5
 5061 JR Oisterwijk, The Netherlands


TANGSHAN ZHENG TONG EXHIBITION CO., LTD.
 NO.24 Chaoyang Road, Lubei District,
 Tangshan City, Hebei Province, China

EU REP

SUNGO Cert GmbH.
 Harffstraße 47,
 40591 Düsseldorf, Germany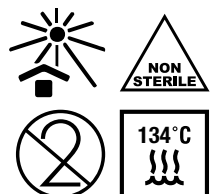

en

Indications:

It plays the role of isolation or auxiliary restoration for oral class II cavity treatment.

Composition:

This product is made of thin stainless-steel sheet.

Instructions for use:

1. First select the appropriate model of band according to the cavity shape and insert the bands by placement forceps in the interdental gap at the edge of the tooth cavity.
2. Put in a suitable wedge, and clamp, make matrix band fit the tooth and avoid both leak and resin filled overhang.

Precautions:

1. This product is used for oral treatment by dentists or health care professionals.
2. If the surface of the matrix bands is found unclean during use, please stop using it.
3. Please pay attention to the matrix bands edge during use to prevent damage to mucosa and gums.
4. Disinfect or sterilize with high temperature and high pressure before use and note that the product packaging box cannot be sterilized with high temperature and high pressure. Dispose as medical waste after use.
5. This is a single-use device.

Storage:

Do not expose to the sun and avoid mixing with corrosive articles.

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